**Ceramco® 3**

**Improved Margin Porcelain and Final Margin  
Instructions**

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Distributed by:  
DENTSPLY Canada  
161 Vinyl Court  
Woodbridge, Ontario  
L4L 4A3  
905-968-5374

**CE**  
0086  
**Rx only**

**DENTSPLY  
CERAMCO**

*The chemistry of a great smile.™*

**Rx only**

# Ceramco<sup>3</sup> Improved Margin Porcelain

*The porcelain margin can be an integral part of a premium restoration, maximizing esthetics and oral health. The porcelain margin helps avoid metal visibility, tissue discoloration and possible overbuilding at the gingiva. A patented carrier medium developed for Ceramco3 margin porcelain facilitates esthetic, easy, accurate and rapid placement for predictable - and profitable porcelain margins.*

## INDICATIONS

The Ceramco3 veneering porcelains may be used for all single and multiple unit porcelain fused-to-metal fixed prosthodontics using conventional high-fusing ceramic alloys with or without silver.

## CONTRAINDICATIONS

Only the indications listed above are suitable.

## WARNINGS/ADVERSE REACTIONS

Consult Ceramco3 MSDS (Material Safety Data Sheets) for safe handling and usage. Use only with recommended Ceramco 3 system components. Use protective equipment to guard against dust when grinding.

## PRECAUTIONS

Refer to the special notes throughout the instructions for best practices for using Ceramco3 materials.

## INSTRUCTIONS

## GENERAL PRINCIPLES

1. Dispense only enough material for immediate use to avoid waste. Dried Ceramco3 margin porcelain prepared with the Ceramco3 Improved Margin and Opaque Correction Liquid can be reused. Select the correct margin porcelain from the following chart. Mixing some of the margin modifier porcelain into the shaded margin porcelain can easily change the porcelain shades.

Margin Porcelain	Corresponding Shade	Ceramco <sup>3</sup> C3 Shades Series	Keyed to Vita <sup>®</sup> 3D Master <sup>®</sup> Shades <sup>2</sup>
i1	i1 , i3		
i2	i2 , i4		
i5	i5, i6, i7, i8		
A1	A1, A2	C3-101	1M1
		C3-102	1M2
		C3-206	2M2
		C3-209	2R2.5
A3	A3, A3.5 B3 D3, D4	C3-204	2L2.5
		C3-207	2M3
		C3-311	3L2.5
		C3-313	3M2
		C3-314	3M3
		C3-315	3R1.5
		C3-316	3R2.5
		C3-419	4M1
		C3-422	4R1.5

Margin Porcelain	Corresponding Shade	Ceramco <sup>3</sup> C3 Shades Series	Keyed to Vita <sup>®</sup> 3D Master <sup>®</sup> Shades <sup>2</sup>
A4	A4 B4	C3-418	4L2.5
		C3-420	4M2
		C3-421	4M3
		C3-423	4R2.5
B2	B1, B2	C3-525	5M2
		C3-526	5M3
		C3-203	2L1.5
C1	C1, C2 D2	C3-205	2M1
		C3-208	2R1.5
		C3-312	3M1
C3	C3, C4	C3-310	3L1.5
		C3-417	4L1.5
		C3-524	5M1

<sup>1</sup>Vita<sup>®</sup> and 3D-Master<sup>®</sup> are registered trademarks of Vita Zahnfabrik H. Rauter GmbH & Co. <sup>2</sup>Data on file.

2. Clean brushes with water after each margin application to prevent contamination.
3. Complete the 5 minute dry and 5 minute preheat cycles to remove organic material and avoid discoloration. For furnaces without a "preheat", dry for 10 minutes.
4. Fire the porcelain on honeycomb sagger trays with metal pins.
5. Properly fired Ceramco3 margin porcelain will have a shiny yet grainy appearance. Firing the dentin porcelain higher than the margin porcelain will result in rounded margins.
6. Be sure the fired porcelain has the correct visual indicator.

## **TOOTH MARGIN PREPARATION**

1. A 90° shoulder preparation or deep chamfer is recommended.
2. Normal tooth reduction (1.5mm) is recommended.

## **SUBSTRUCTURE DESIGN**

Extend the facial surface of the metal substructure so that the metal ends at the axial gingival corner of the margin preparation or extends slightly onto the gingival ledge.

*Proper Metal Design*

## **OPAQUE PROCEDURE**

Apply and fire the Ceramco3 paste opaque or Ceramco3 powder opaque porcelain using your normal procedures.

Be sure the opaque completely covers the metal at the gingival margin. Failure to do so will make the completed porcelain margin appear gray.

## DIE SEALER APPLICATION

1. Using the Ceramco3 die sealer, apply a thin, even layer of sealer over the die margin area. Blow off any excess die sealer and allow the liquid to thoroughly dry.
2. Reapply the die sealer at least 2 more times and allow to dry.
3. Mark the facial margin with a wax pencil.

*Die Sealer Application*

## DIE RELEASE APPLICATION

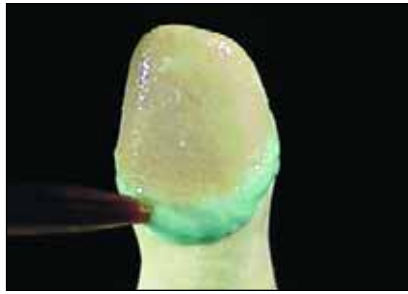
1. Be sure the die margin is clean.
2. Using the Ceramco3 die release, apply a thin even layer of die release over the margin area. Blow off the excess die release.
3. **Apply and allow to dry several more times until the die has a very slight sheen.**

*Die Release Application*

## FIRST MARGIN APPLICATION

1. Place a small amount of the shaded margin porcelain on a clean palette.
2. If modifier is necessary, add the modifier porcelain to the shaded margin porcelain.
3. Mix the Ceramco3 Improved Margin and Opaque Correction Liquid with the Improved Margin Porcelain to a thick creamy consistency. This mixture will air harden in approximately 5 minutes. If additional working time is required, mix in a small amount of distilled water.
4. Place the porcelain mixture in the gingival area, pushing the mixture down to but not over the gingival margin. Do not over build.

*First Shoulder Application*

**Note:** All porcelains shrink by volume. The larger the porcelain mass the more apparent the firing shrinkage. Better results are achieved by applying a small amount of porcelain on the first porcelain application.

5. Do not allow the restoration to become completely dry before lifting off the model.
6. Carefully lift the coping from the die.

*First Shoulder Lift-off*

**Note:** If inspection reveals the need for adjustment, carefully return the coping to the die and add fresh material. It is not necessary to remove the unfired porcelain.

7. Dry and fire the restoration according to the recommended temperatures.
8. The correct visual indicator will be shiny yet grainy.

*First Shoulder Fired*

## SECOND MARGIN APPLICATION

1. Be sure the die is clean and reapply the die release. Allow to dry.
2. Place the coping on the die. Make a new mix of Improved Ceramco3 Margin porcelain and Ceramco3 Improved Margin and Opaque Correction Liquid. Apply the mixture to the gingival margin.

*Second Shoulder Application*

3. Do not allow the restoration to become completely dry before lifting off the model.

*Second Shoulder Lift-Off*

4. Dry and fire the restoration according to the recommended temperatures.
5. The correct visual indicator will be shiny yet grainy.

*Shoulder Margin Completed*

**Note:** If additional applications are required, repeat the procedures used for the second margin application.

# Ceramco<sup>®</sup>3 Final Margin

## FINAL MARGIN APPLICATION

The Final Margin is designed to be a final porcelain margin application that can be used to repair small margin imperfections. It is designed to be fired at the same time as the glaze cycle.

Final Margin Porcelain	Corresponding Shade	Ceramco <sup>®</sup> 3 C3 Shades Series	Keyed to Vita <sup>®</sup> 3D Master <sup>®</sup> 1 Shades <sup>2</sup>
i1	i1 , i3		
i2	i2 , i4		
i5	i5, i6, i7, i8		
A1	A1, A2	C3-101	1M1
		C3-102	1M2
		C3-206	2M2
		C3-209	2R2.5
A3	A3, A3.5 B3 D3, D4	C3-204	2L2.5
		C3-207	2M3
		C3-311	3L2.5
		C3-313	3M2
		C3-314	3M3
		C3-315	3R1.5
		C3-316	3R2.5
		C3-419	4M1
		C3-422	4R1.5

Final Margin Porcelain	Corresponding Shade	Ceramco <sup>®</sup> 3 C3 Shades Series	Keyed to Vita <sup>®</sup> 3D Master <sup>®</sup> 1 Shades <sup>2</sup>
A4	A4 B4	C3-418	4L2.5
		C3-420	4M2
		C3-421	4M3
		C3-423	4R2.5
		C3-525	5M2
		C3-526	5M3
B2	B1, B2	C3-203	2L1.5
C1	C1, C2 D2	C3-205	2M1
		C3-208	2R1.5
		C3-312	3M1
C3	C3, C4	C3-310	3L1.5
		C3-417	4L1.5
		C3-524	5M1

<sup>1</sup>Vita<sup>®</sup> and 3D-Master<sup>®</sup> are registered trademarks of Vita Zahnfabrik H. Rauter GmbH & Co. <sup>2</sup>Data on file.

1. Be sure the die is clean
2. Reapply the die release and allow to dry
3. Place the coping on the die and apply the Final Margin porcelain
4. Allow to dry several minutes, then lift the coping off the die. DO NOT ALLOW TO DRY COMPLETELY before lifting off the die.
5. If using an overglaze, carefully apply the glaze to the restoration.
6. Fire the restoration according to the recommended temperatures.

# FIRING REFERENCE CHARTS

## FIRING PROGRAM IN F°

Program Description	Time (min)			Time (min)			Vacuum	Set Temp		Temperature			
	Dry	Pre Heat	Vac Hold	Hi Temp Hold	Hold	Cool	Set Point (in Hg)	Idle	Hi Temp	Vac Start	Vac Stop	Heat Rate F°/min	Night
Paste Opaque	5	3	0	0	0	0	29	932	1787	932	1787	180	212
Powder Opaque	3	3	0	0	0	0	29	1202	1778	1202	1778	126	212
Margin-Improved	5	5	0	1	0	0	29	1202	1769	1202	1769	126	212
Final Margin (with overglaze)	3	3	0	0.5	0	0	0	1202	1715			126	212
Final Margin (with natural glaze)	3	3	0	0.5	0	0	0	1202	1733			126	212
Op Dentin/Dentin/Mod/Enamels	5	5	0	0	0	0	29	1202	1760	1202	1760	99	212
Natural Glaze	3	3	0	0.5	0	0	0	1202	1733			126	212
Overglaze	3	3	0	0.5	0	0	0	1202	1715			126	212
Add-On	5	5	0	0	0	0	29	1202	1724	1202	1724	126	212

## FIRING PROGRAM IN C°

Program Description	Time (min)			Time (min)			Vacuum	Set Temp		Temperature			
	Dry	Pre Heat	Vac Hold	Hi Temp Hold	Hold	Cool	Set Point (in Hg)	Idle	Hi Temp	Vac Start	Vac Stop	Heat Rate C°/min	Night
Paste Opaque	5	3	0	0	0	0	29	500	975	500	975	100	100
Powder Opaque	3	3	0	0	0	0	29	650	970	650	970	70	100
Margin-Improved	5	5	0	1	0	0	29	650	965	650	965	70	100
Final Margin (with overglaze)	3	3	0	0.5	0	0	0	650	935			70	100
Final Margin (with natural glaze)	3	3	0	0.5	0	0	0	650	945			70	100
Op Dentin/Dentin/Mod/Enamels	5	5	0	0	0	0	29	650	960	650	960	55	100
Natural Glaze	3	3	0	0.5	0	0	0	650	945			70	100
Overglaze	3	3	0	0.5	0	0	0	650	935			70	100
Add-On	5	5	0	0	0	0	29	650	940	650	940	70	100

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